

Cleaning Guide

This guide incorporates all individual cleaning instructions for the telecare-based product range as given in user documentation. The texts are extracts from the manufacturers' guidelines.

PRODUCT NAME	PART NUMBER	CLEANING INSTRUCTIONS
All products not mentioned below		Cleaning can be done with a soft, damp cloth with water or domestic mild detergent.
Lifeline 4000/400	All variants	Dust with a soft cloth which can be moistened with a gentle detergent if required.
Lifeline Gateway	36900/50	The unit should be cleaned with a dry lint free cloth.
Temperature Extremes Sensor	69005/03 and 69005/05	Should be cleaned approximately every six months to ensure enclosure remains free of dust, cobweb and grease. Dust with a dry cloth or vacuumed, do not use water or a damp cloth.
Enuresis Sensor	41005/21	Clean the mat with a little general-purpose detergent and a non-abrasive damp cloth. Dry with a suitable non-abrasive cloth.
Epilepsy Sensor	67005/59	Clean the mat with a general-purpose detergent, ALWAYS dry after cleaning. Clean the control unit with a moist cloth.
Smoke/Heat Detector (Ei)	9000/77	Clean regularly, particularly in dusty areas. Use a narrow nozzle attachment of a vacuum cleaner to remove dust, insects and cobwebs from the sides and cover slots where the smoke or heat enters. To clean the cover, wipe with a damp cloth and dry thoroughly with a lint free cloth.
Smoke Detector (BRK)	9000/30	Clean at least once per month; gently vacuum the outside of the alarm using a household vacuum with soft brush attachment. Never use water, cleaners or solvents since they may damage the unit.
Smoke Detector (Detectomat)	69005/74	Remove from base and clear away any dust, wipe with a damp cloth.
Natural Gas Detector (4WARN)	GS831	Occasionally wipe with a clean damp cloth. Do not use cleaning agents, bleach, polish, alcohol derivatives or composites.
Natural Gas Detector (SF Z10)	GS783	To clean wipe with a dry cloth. Do not use cleaning agents, bleach or polish.
Fall Detector	67005/49	Clean periodically with a damp cloth and mild detergent.
Gas Controller	GS807	Clean periodically with a damp cloth. Do not use bleach, aerosol sprays or other cleaning agents.
CO Detector	67005/51	Clean the outside with tissue and ensure holes on the front are not blocked. Do not use cleaning agents, bleach or polish.