

# CareAssist

## What is the CareAssist?

The CareAssist is a portable telecare alarm that can receive telecare alerts and display not only the type of sensor activated but also either the location of the sensor and/or the name of the person the sensor has been assigned to. This means that onsite carers can be quickly made aware of any incidents allowing them to provide a high level of care whilst maximising their efficiency. It will also help to reduce the impact on informal carers' lifestyles enabling them to lead more independent lives.



## How does it work?

Telecare sensors are pre-assigned to the CareAssist and allocated to either a location within a building e.g. communal lounge or to a specific resident e.g. Mrs Smith Flat 1. When the telecare sensor is activated, the CareAssist will vibrate, beep and light up its bright screen to alert the carer to the incident allowing them to view what sensor activated the alarm, which room/resident it is allocated to and the time and date of the alarm enabling them to take speedy and appropriate action.

## Who is it for?

CareAssist is designed to support local carers to deliver high quality, non-intrusive care. It is extremely easy to use, portable device that provides carers working or living onsite with a means to receive instant alerts from Tunstall's range of telecare sensors.

As a result, onsite carers can be quickly made aware of any incidents and provides a very cost effective telecare solution whilst avoiding the need for a telephone line or monitoring centre service which is often not required when full time (24 hour) care is being provided.

With a radio range of up to 200 metres (line of sight), it is particularly suited to small specialist dementia care, intermediate care, learning disability, residential & care home, and respite care facilities.

It can also provide significant benefits to private individuals and their informal carers who may prefer not to be connected to a 24 hour monitoring centre service.

## Features and benefits

- Compatible with Tunstall telecare range - Supporting up to 86 telecare sensors
- Easy to use - with clear luminous 2.4" OLED display
- Fully portable - small pocket sized design and battery operated
- Multiple alert modes - the unit can alert the carer using audible alerts, on screen message and a vibration mode. The audible alerts and vibration mode can be turned off if necessary.

## Technical details

Weight:	148g
Dimensions:	128 x 73 x 22mm (L x W x D)
Radio frequency:	869.2125MHz
Radio range:	Up to 200m (line of sight), up to 50m (in building)
Battery Backup Time:	Up to 12 hours
Battery Charge Time:	Up to 2 hours (requires initial 24 hour charge)
Programmable Sensors:	Up to 86
Number of locations/users:	139 locations (inc 64 custom locations)
Memory capacity:	Last 256 events
Warranty:	2 years
Accessories supplied:	Belt clip, USB charger, Antenna, User Guide

## Standards

CE:	Compliant	
Part number	CareAssist	68100/01

The following accessories are also available:

### Accessory:

Part Number	PC download lead (RS232):	D6705015A
	RS232 to USB converter:	S8502004

### Replacement Accessories:

Part Number	Battery	S1004037
	Belt clip	D6702098A
	Antenna	S2005018
	Micro USB charger	XD6706011A

### Pager System Accessories

Part Number	Pager	i9000/69A
	Pager Transmitter*	i9000/61A
	Pager Transmitter Connection lead (Connects CareAssist to the Transmitter)	D6705015A

Our policy of continual development means that product specification and appearance may change without notice. Tunstall does not accept responsibility for any errors and omissions contained within this document.

© 2012 Tunstall Group Ltd.

\* TUNSTALL is a registered trademark.

Tunstall Healthcare (UK) Ltd,  
Whitley Lodge, Whitley,  
Yorkshire DN14 0HR

Tel: 01977 661234  
Fax: 01977 662570

Email: [enquiries@tunstall.com](mailto:enquiries@tunstall.com)  
tunstall.com



The Tunstall logo, consisting of the word 'Tunstall' in white, bold, sans-serif font on a red rectangular background.